

Topi-CLICK®

Topical Dosing Applicators

by DOSELOGIX

Topi-CLICK MICRO®




Topical Micro-dosing Application

Veterinary Patient Instructions

- ✓ Automated, metered topical dosing provides an accurate and consistent dose every time
- ✓ Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Innovative TRIAD shape allows for precision handling
- ✓ Built-in REFILL Badge helps pet owners reorder medication on-time




STEP 1:
Unscrew the cap of the Topi-CLICK Micro® and set aside for recapping after use.

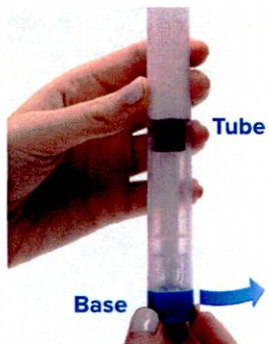

STEP 2:
Holding the Micro® in an upright position, turn/CLICK the **Base** the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile, and Visual CLICK points of the triad shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube** to ensure dose is fully dispensed.



STEP 3:
Apply topical medication with built-in applicator as directed by veterinarian or pharmacist. **Continue rubbing until preparation has completely saturated the prescribed areas through your pet's fur.** **IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply medication.

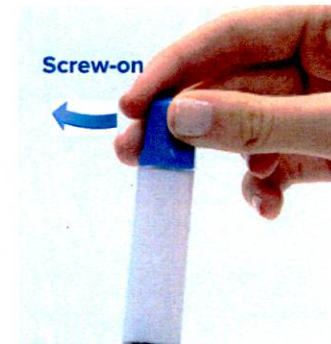

STEP 4:
Screw on cap after each dose to prevent evaporation. Be sure cap is secured completely.

Innovative TRIAD Shape Allows Precision Application Control

Each turn/CLICK of the base dispenses 0.05 mL of medication accurately.

REFILL Badge™



REFILL Line

Plunger

The Refill Badge™ reminder system helps pet owners reorder in a timely manner.

When the **TOP** black line of the **Plunger** reaches the embossed line below the word "REFILL" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "REFILL".

Note: Follow instructions on prescription label before use. Topi-CLICK Micro® is not re-fillable.

Made of FDA grade materials | Phthalate free | blocks UV and is Recyclable ♻️.

The Best User Experience™

DoseLogix.com | 877.870.8448 | info@DoseLogix.com

